

Operation Manual

Massage Table

61175-61177

toboli



Illustration similar, may vary depending on model

Read and follow the operating instructions and safety information before using for the first time.

Technical changes reserved!

Due to further developments, illustrations, functioning steps, and technical data can differ insignificantly.

Updating the documentation

If you have suggestions for improvement or have found any irregularities, please contact us.



The information contained in this document may alter at any time without previous notice. It is prohibited to copy or spread any parts of this document in any way without prior written allowance. All rights reserved.

The WilTec Wildanger Technik GmbH cannot be held accountable for any possible mistakes in this operating manual, nor in the diagrams and figures shown.

Even though, the WilTec Wildanger Technik GmbH has undergone biggest possible efforts to ensure that the operating manual is complete, faultless, and up to date, mistakes cannot be entirely avoided.

If you should find a mistake or wish to make a suggestion for improvement, we look forward to hearing from you.

Send an e-mail to:

service@wiltec.info

or use our contact form:

<https://www.wiltec.de/contacts/>

The most recent version of this manual in various languages can be found in our online shop via:

<https://www.wiltec.de/docsearch>

Our postal address is:

WilTec Wildanger Technik GmbH
Königsbenden 12
52249 Eschweiler
Germany

To return orders for exchange, repair, or other purposes, please use the following address. Attention! To allow for a smooth execution of your complaint or return, it is important to contact our customer service team before returning the goods.

Returns Department
WilTec Wildanger Technik GmbH
Königsbenden 28
52249 Eschweiler

E-mail: **service@wiltec.info**

Tel: +49 2403 55592-0

Fax: +49 2403 55592-15

Introduction

Thank you for purchasing this quality product. **To minimise the risk of injury we urge that our clients take some basic safety precautions when using this device. Please read the operation instructions carefully and make sure you have understood its content.**

Keep these operation instructions safe.

Precautionary instructions

- Maintain the table periodically. Make sure all screws are secure and the cable has not deteriorated or frayed. Make sure there are no cracks in the wood structure.
- Do not overload the table beyond the recommended weight limit. This could cause damage to vinyl materials. Use warm soapy water and wipe dry.
- Do not use alcohol-based cleaners when cleaning the vinyl. This could cause damage to vinyl materials. Use warm soapy water and wipe dry.
- Make that all four legs are on a solid surface and at the same level when using the table.

Table set-up

- Place the table on its side, release the buckles, partially open the table, and remove the accessories.
- Standing behind the table, grasp both table handles and gently swing in an outward motion to fully open the table. The legs should open automatically.
- Turn the table upright by lifting the table. Press down in the middle of the table to make sure the table lies flat. If the table peaks in the middle, check and straighten out the cable.

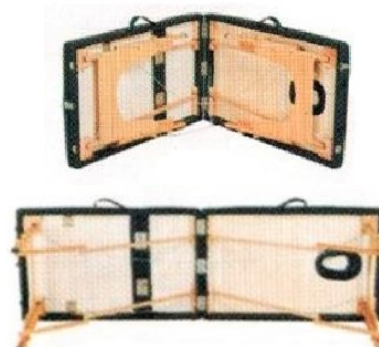


Table and accessory use

We provide optional accessories for you. Please read the instructions carefully before use.

Headrest

The headrest consists of a half-moon shaped headrest base and half-moon shaped face pad. It will give your client comfortable head and face support. You may have a different option for the headrest base. The adjustable headrest is easy to hang. Change the angle and height of the base by adjusting the base handle.



Adjustable armrest shelf

This adjustable armrest shelf is a good device for supporting your client's arms at the front of the table. Hang the armrest shelf on the two bars of the headrest and slide the belt up or down for the optimal position.



Side arm extension

Your new massage table features a side arm extension. These removable padded side armrests add table width. They fit into the required accessory holes in each side of the table. That will give your clients comfortable arm support, especially for larger clients.

Height adjustment

Proper table height is determined by the height of the therapist, massage manner and technique. You can adjust the height by removing the grip knob from each leg. Replace the knobs after adjusting each leg to the desired position.

Table weight limits

- Positioning the client in the centre of the table will help to distribute the weight evenly and ensure stability. Gently sit near (not on) the centre of the table at first, then sit as close to the centre (from the edge) as possible. The working weight is the combined weight of the client and the maximum amount of downward pressure being applied by the therapist. The maximum working weight for the table is 150 kg.
- Example: For a client weighing 80 kg, the maximum downward pressure that can safely be applied is 70 kg. As pressure is applied at a sideward angle puts considerably more stress on the table, the maximum sideward pressure should not exceed 50 % of the maximum downward pressure.
- **DO NOT EXCEED REFERRED MAXIMUM WORKING WEIGHT!** Never exceed the load limits of the table! Doing so may result in serious injury and/or damage!

Storage

- Attach headrest base and face pad onto the elastic under the table as shown. Place each accessory in its original position as shown in the figure.
- The carry case should be used when transporting or storing the table to help protecting the table from any possible damage. When not in use, store the table in a cool, dry place and away from direct sunlight. Do not expose the table to extreme (hot or cold) temperatures or moisture.



Table case



Care

Upholstery

- **Daily Cleaning:** Remove daily dirt and grime from the table with a mild soap and warm water solution. Dry with a soft, lint-free cloth.
- **Disinfecting:** Disinfect the table as required with an isopropanol-based hospital grade disinfectant cleaner (approved for use on polyurethane vinyl).
- **Stain removal:** Some minor stains can be removed with a mild, non-abrasive cleaner. Spray a damp cloth with cleaner and gently wipe to remove any stains, rinse thoroughly with warm water, and dry it with a soft lint-free cloth. More stubborn stains may require a professional cleaning.

Wood care

Your massage table is made of selected hardwood and is finished with lacquer so that it does not require special care. If desired you may polish occasionally with quality wood care products.

Headrest and arm shelf installation instructions

1. Put the headrest tubular arms through the loops of the two arm shelf straps.



2. Insert the tubular arms into the headrest holes in the table and lock the position using the locking lever of the headrest.



3. Adjust the arm shelf to the desired horizontal position and tighten all the straps accordingly.

4. Attach the headrest pillow to the headrest with the provided Velcro fastener.



Using the 6-way adjustable headrest

1. Put the tubular arms of the face cradle into the ports.
2. Position the lock lever so that it extends out horizontally with the ridges on top.



3. Adjust the headrest as desired and then move the lever downwards to lock into place. Keep the table dry and avoid exposure to wet or humid environments. Should the table get wet, quickly and thoroughly dry with a soft cloth.

Back plank height adjustment

- After removing the one buckle at the end you can raise the back plank. Adjust it to a desired position.
- After you have adjusted the back plank to maximum height, place the buckles back on.



Important notice:

The reprint or reproduction, even of excerpts, and any commercial use, even in part, of this instruction manual require the written permission of WilTec Wildanger Technik GmbH.